

Endo Reality Training Teeth

- Instructions for use -

Please follow the below instructions to maximise your endodontic experience:

- Water can be used as an irrigant or sodium hypochlorite if preferred.
 - The teeth can be accessed using a high speed handpiece.
 - Ultrasonic tips can be used in a wet environment.
 - Normal endodontic rotary settings can be used.
 - Warm vertical obturation techniques work ideally at 150°C.
- Radiographs should be taken using a short exposure time and the lowest kV.



ENDO REALITY LTD 10703639

Tel 07818 401 050 Web www.endoreality.co.uk

102 Bowen Court, St Asaph Business Park, St Asaph, LL17 0JE