

Endo Reality Jaw - Information -

1. Simulated gum varies in rigidity in hot and cold environments.
2. Please dry Endo Reality Jaw after use and place upside down to drip dry. This will prolong longevity of the simulated gum.
3. Endo Reality Jaw is not designed to be long lasting. The simulated gum will deteriorate and rip over time.
4. Endo Reality Jaw fits most phantom heads using the universal bolt provided.
5. Anterior teeth and teeth with conical roots will require digital pressure when using rotary instruments to prevent them from rotating.



ENDO REALITY LTD 10703639

Tel 07818 401 050 Web www.endoreality.co.uk

102 Bowen Court, St Asaph Business Park, St Asaph, LL17 0JE