

## Endo Reality Mini Trainer

- Instructions for use -

1. Place tooth into the appropriate size upper compartment. Incisors, canine and premolars fit into the small locator and molars into the larger locator.
2. Fill the base with Endo Reality conductive gel, slightly overfilling to ensure when the upper compartment is placed, the tooth roots are in contact with gel.
3. Connect appropriate upper compartment with tooth inserted, to base.
4. Connect apex locator to the mini trainer.
5. Ensure the canal is dry before using apex locator to determine apex reading.
6. Over instrumentation or large open apices may give an inaccurate reading.
7. Clean out gel and dry mini trainer at end of use.
8. Gel may cause skin irritation, gloves are advised when handling.

